



EASY INSTRUMENT PRE-TREATMENT

Aniosyme Foam

READY-TO-USE PRE-TREATMENT FOAM FOR MEDICAL AND SURGICAL INSTRUMENTS

Foam spray for cleaning of medical and surgical instruments. Ensures optimised dissolving of contamination on instruments with longer standing time (up to 72 hours) or before machine processing.

Aniosyme Foam is:

- **EASY** - Simple to use
- **FAST** - Prevents drying out of contamination
- **EFFECTIVE** - Stable foam with good wetting



Aniosyme Foam

Ready-to-use pre-treatment foam for medical and surgical instruments



Aniosyme Foam
Spray

EASY

Simple to use.

FAST

Prevents drying out of contamination.

EFFECTIVE


Stable foam with good wetting.

OUR OFFER	YOUR BENEFITS	ANIOSYME FOAM
Easy - Simple to use	Ready-to-use solution in a handy bottle	Yes
Fast - Prevents drying out of contamination	Makes further preparation easier	Yes
Effective - Stable foam for good wetting	Preparation can start even during surgery	Yes

AREAS OF USE

Ready-to-use pre-treatment foam for optimised dissolving of contamination on instruments with longer standing times (up to 72 hours) or before machine processing. Prevents drying out of contamination and makes pre-cleaning easier with a good cleaning effect. The thick foam guarantees even coverage of the instrument surface. Aniosyme Foam is compatible with a wide variety of materials. Not suitable for instruments made of zinc or iron.

INSTRUCTIONS FOR USING ANIOSYME FOAM

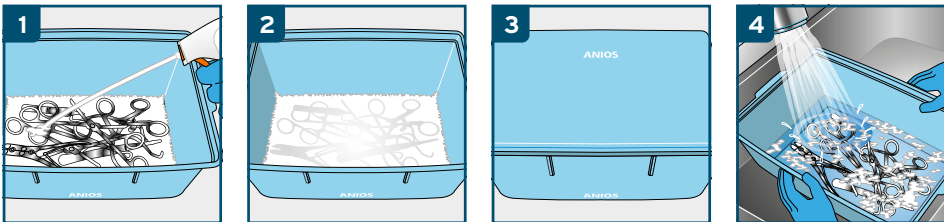

1. Beforehand, spray the bottom of the soaking tray with Aniosyme Foam and place instruments collected.
2. Re-cover all the instruments with foam.
3. Cover the tray with the lid.
4. Thoroughly rinse treated medical devices with tap water to remove all traces of product and organic soiling.

ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
Aniosyme Foam	12 x 750 ml bottles	30 914 20 PZN 138 795 83



CE LABORATOIRES ANIOS
1, Rue de l'Espoir
59260 Lezennes
France

Distributed by:
ECOLAB DEUTSCHLAND GmbH
Ecolab-Allee 1
40789 Monheim am Rhein
Germany

APPLICATION AREA

Cleaning of medical-surgical instrumentation: for dry transport or before machine cleaning. Maintains an adequate level of humidity to avoid blood and other organic materials drying.

INSTRUCTIONS FOR USE

Ready-to-use solution.

1. Beforehand, spray the bottom of the soaking tray with Aniosyme Foam and place instruments collected.
2. Re-cover all the instruments with foam.
3. Cover the tray with the lid.
4. Thoroughly rinse treated medical devices with tap water to remove all traces of product and organic soiling.

Not suitable for instruments made of zinc or iron.

INGREDIENTS

Non-ionic tensides, enzymes (proteases, lipases and amylases), complexing agents, excipients.

STORAGE

Dangerous - respect the precautions for use (Drawn up according to the European rules in force regarding the classification and labelling of chemical products). Storage: between +5°C and +25°C.

CERTIFICATES

On demand we are pleased to provide you with certificates.

MEDICAL DEVICE USE SAFELY

Please read label and product information.



FOR PROFESSIONAL USE ONLY.

